Greifzange | Folding Reacher

Art. 81 27 21

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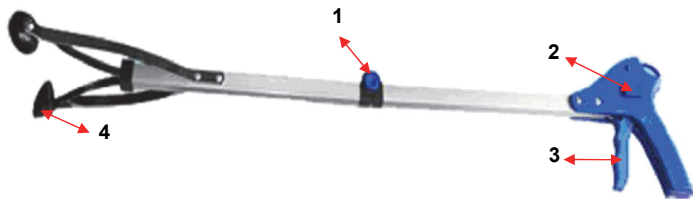
Safety Notes

- The folding reacher is not a toy. Keep the article out of reach of children.
 - The folding reacher can grab objects up to a maximum weight of one kilogram. Do not exceed the maximum weight.
 - Before using the folding reacher make sure the unit is unfolded and properly locked.
-

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Using

1. Unfold the reacher and press the lock button (1) from the side “PUSH TO LOCK” inwards.
2. Push the slide switch (2) away from the handle in order to release the trigger (3).
3. Press the trigger (3) to grab the desired object. Keep the trigger pressed until both rubber cups (4) have fixed the object securely. While keeping the trigger (3) pressed, push the slide switch (2) in direction to the handle in order to engage the trigger in this position.
4. To release the trigger, lightly press the trigger inwards and push the slide switch in opposite direction.
5. If you do not use the folding reacher, leave the rubber cups (4) open and the trigger unlocked.

Cleaning and Storing

- Clean the folding reacher with a cloth. Do not use aggressive chemicals.
- If desired, fold down the folding reacher for space-saving storage. Press the lock button (1) from the side “PUSH TO UNLOCK” inwards and fold down.